




---

# Sheath Fitting Guide




# Sheath Fitting Guide


A step by step guide to correctly fitting your Discreet Sheath.

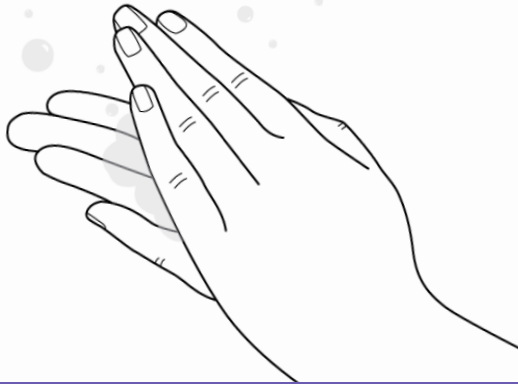


1. Prepare by washing and drying the anatomical area and thoroughly washing your hands.

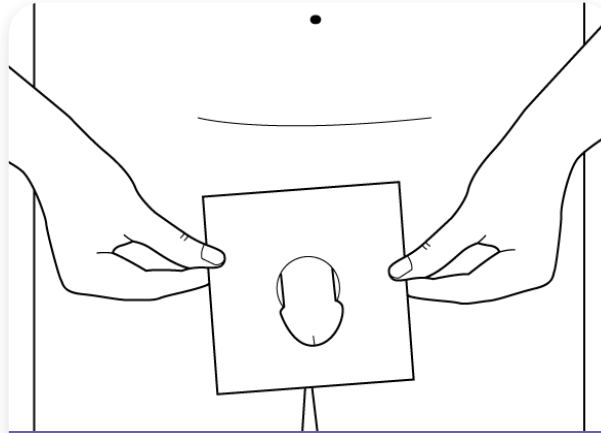


2. Trim pubic hair or if preferred use a hair protector\*.  
\*Not included

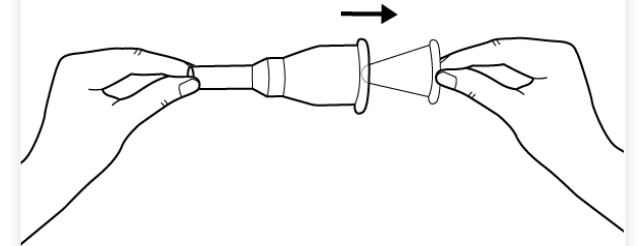


3. Remove the Discreet Sheath from the packaging, and remove from cone if appropriate.

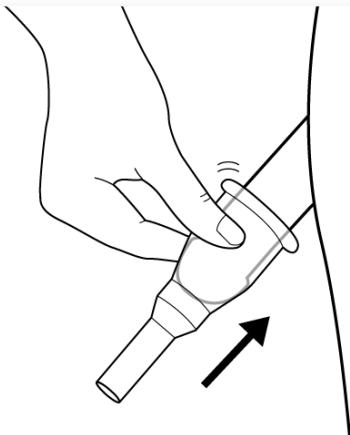

4. Unroll the Discreet Sheath over the penis and hold in place for 10 seconds.

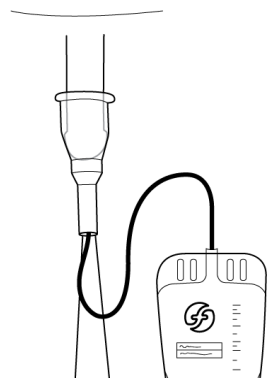

5. Connect the Discreet Sheath to the drainage Bag.

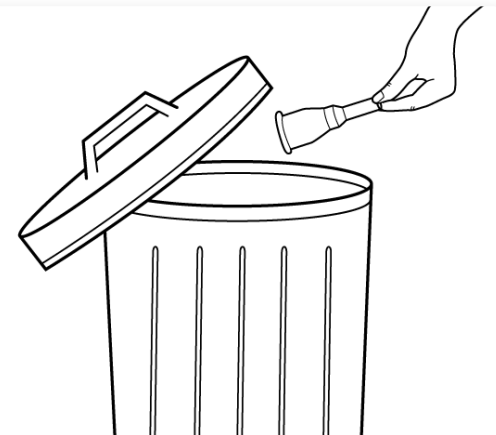

6. Replace the Discreet Sheath every 24 hours. To remove use warm soapy water or adhesive remover, gently roll the sheath off the penis and discard.


# All you need for your comfort and protection in 3 easy steps:



## 1. Discreet Sheath Urinary Catheter

REGULAR	TACKY / WIDE	SHORT	DESCRIPTION	QTY
00-DSR25	00-DST25	00-DSS25	25mm Discreet Sheath Urinary Catheter	30
00-DSR29	00-DST29	00-DSS29	29mm Discreet Sheath Urinary Catheter	30
00-DSR32	00-DST32	00-DSS32	32mm Discreet Sheath Urinary Catheter	30
00-DSR36	00-DST36	00-DSS36	36mm Discreet Sheath Urinary Catheter	30
00-DSR41	00-DST41	00-DSS41	41mm Discreet Sheath Urinary Catheter	30


Click or scan to view  
our range of Discreet  
sheath

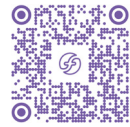

## 2. 500ml Leg Bag

PART NUMBER	DESCRIPTION	QTY
00-1502	Leg Bag 500ml with Fabric Backing - Short Tube	10
00-2502	Leg Bag 500ml with Fabric Backing - Long Tube	10

Click or scan to view  
our 500ml Leg Bags

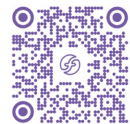

## 3. 2Ltr Night Bags

SHORT	DESCRIPTION	QTY
00-2203c	F2-EZ 2L Once Drainable Bag with Twist-Snap Top Tap Non-Sterile	10
00-2208c	F2-EZB 2L Once Drainable Bag with Bottom Twist-Snap Tap Non Sterile	10

Click or scan to view  
our 2Ltr Night Bags



### Flexicare Medical Limited

Cynon Valley Business Park • Mountain Ash • CF45 4ER • UK  
+44 (0)1443 474647 • enquiries@flexicare.com • www.myflexicare.com



FC82641A | 052024